



**A Walsh  
Imaging Inc.**

973-616-7100 • Fax 973-616-7191  
www.awalshimaging.com

Facility: \_\_\_\_\_ Dept: \_\_\_\_\_ Processor: \_\_\_\_\_

### Semi-Annual Radiographic Artifact Evaluation

		Date:	
		Initials:	
Scratches	Dirt on counter top or feed tray		
	Dirty or damaged processor rollers/ guide shoes/ turnaround		
	Stuck film which is scratching other film.		
Fingerprints & smudges	Wet or dirty hands when handling film.		
Crinkle marks (finger nail)	Crescent shaped mark resulting from bending the film while loading or unloading the film.		
Partially exposed or fogged film	Light leak in darkroom or cassette.		
	Scatter radiation to cassette left in x-ray room during exposure		
Static	Relative humidity is low and clothing generates static electricity which discharges onto film during handling		
Black spots	Film wet or contaminated prior to processing		
	Emulsion from previous films transfers from dirty processor roller to film		
White spots	Dirty screens		
	Emulsion pulled from film by dirty processor rollers		
	Defect in film emulsion		
Brown spots	Inadequate washing		
	Inadequate fixing		
Water spots	Depleted chemicals in processor		
	Poor squeegee action at wash rack exit		
	Clogged dryer air tubes		
Other			

P = PASSED F = FAILED N/A = DOES NOT APPLY