

Facility:	Department:	X-Ray Unit:
•	-	•

Quarterly X-Ray and Light Field Alignment

Initials:

Date:

Place a cassette table top. Mark the edges of the light field with a penny on all four sides. Expose and process the film. Measure the difference between the edge of the x-ray field and the edge of the coin marking the light field.

	vel 2% of SID D)(.02)			
Side 1	Long side	Difference		
Side 2	Long side	Difference		
	Add Diff	erence 1 & 2 =	Is this less than tolerance level?	Y N
Side 3	Short side	Difference		
Side 4	Short side	Difference		
	-		 ls this less than	

If answers to A & B are both Yes the test results Pass If either A or B is no the unit Fails. (<i>Corrective action must be completed in 30 Days</i>)		Fail
--	--	------